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A. Intended use

Cito NovaClean 25% And Cito NovaClean 50% solutions are high-level heat disinfectants based on citric acid, used for simultaneous disinfection and decalcification of dialysis machines. These solutions are effective against bacteria, fungi, and viruses.

B. Advantages

- effective against a wide range of microorganisms (including fungi EN1650, bacteria EN1276, and viruses ISIRI16676, ISIRI17981).
- This disinfectant solution is designed to clean and sanitize by removing both particulate matter (sediment) and dissolved substances like salts and organic matter from blood. It also effectively eliminates contaminants and inhibits biofilm formation.
- It does not cause corrosion or damage.
- Biodegradable and free of environmental effects.
- Easily rinsed from dialysis circuits with no toxic or allergenic residues.
- Approved by the Iranian Food and Drug Administration and certified for GMP compliance.

C. Compositions

Citric acid, Malic acid, Lactic acid, and Deionized water.

D. Warnings

- To disinfect the hemodialysis machine using CitoNovaClean 25% or CitoNovaClean 50% (see Part E).
- Use within 30 days after opening the container.
- Store in a tightly closed container in a cool, well-ventilated area at 1°C to 30°C, away from direct sunlight.
- Avoid inhalation, ingestion, and direct contact with the concentrated solution. In case of skin or eye contact, rinse immediately with plenty of water and seek medical attention if necessary.
 - Use protective gloves and face masks when handling the solution.

E. Instruction for use

To disinfect the hemodialysis machine using Cito NovaClean, follow the machine manufacturer's instructions for hot disinfection with paracetic acid-

based solutions, or consult certified service or maintenance experts.

F. Manufacturer

Name: Nova Pharmed Chemia (Member of the Iranian Iranian Consortium of Dialysis and JAM TRITA Group)
Factory: 4th Bustan, 4th Golestan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN
Office: 5th floor, No. 44, Saadat Abad Ave., Tehran, IRAN,
Post code: 1998836841.
Tel: +98(21)220 73102 Fac.: +98(26)37779581
Email: info@Novapharmed.co
Website: www.Novapharmed.co

G. Symbols

	Store between 1°C – 30°C
	Attention to cautions
	Read Instructions for use
	Sing up
	NFPA
	Health Hazard/ irritation
	Corrosive substance
	Manufacturer
	Keep away from sunlight
	Keep dry
	Do not open with sharp object