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ICD GROUP

JAM TRITA GROUP

Doc. Code	Rev. No.:	Date Of Issue	Page
QM17RDW009	R02	1404/05/06(28/July. /2025)	1 of 2

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A. Intended use

AcetinoClean 12% is a concentrated acetic acid-based solution that functions simultaneously as an effective disinfectant, descaler, and cleaner, specifically formulated for use in dialysis machines.

B. Advantages

- AcetinoClean 12% is effective against a wide range of microorganisms (including: fungi EN1650, bacteria EN1276, and viruses ISIRI16676, ISIRI17981).
- Compatible with all acetic acid-based hemodialysis machines; does not cause corrosion or damage.
- Environmentally friendly. Easily rinsed from dialysis circuits with no toxic or allergenic residues.
- Approved by the Iranian Food and Drug Administration (IFDA) and certified for GMP compliance.

C. Compositions

Acetic acid, Deionized water.

D. Warnings

- Ensure the container is intact.
- Use within 30 days after opening the container.
- Store in a tightly closed container, in a cool, well-ventilated area at 15°C to 30°C, away from direct sunlight.
- Avoid inhalation, ingestion, and direct contact with the concentrated solution. In case of direct contact with the concentrated product, rinse immediately with plenty of water and seek medical attention if necessary.
- Use protective gloves and face masks when handling the solution.

E. Instruction for use

To disinfect the hemodialysis machine using AcetinoClean 12%, follow the machine manufacturer's instructions for cold disinfection with acetic acid-based solutions, or consult certified service or maintenance experts.

F. Manufacturer

Name: Nova Pharmed Chemia (Member of the Iranian Iranian Consortium of Dialysis and JAM TRITA Group)

Factory: 4th Bustan, 4th Golestan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN

Office: 5th floor, No. 44, Saadat Abad Ave., Tehran, IRAN, Post code:1998836841.

Tel: +98(21)220 73102

Fac.: +98(26)37779581

Email: info@Novapharmed.co

Website: www.Novapharmed.co

G. Symbols

	Store between 15°C – 30°C
	Attention to cautions
	Read Instructions for use
	Sing up
	NFPA
	Health Hazard/ irritation
	Corrosive substance
	Manufacturer
	Keep away from sunlight
	Keep dry
	Do not open with sharp object

Doc. Code	Rev. No.:	Date Of Issue	Page
QM17RDW009	R02	1404/05/06(28/July/2025)	2 of 2