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Doc. Code	Rev. No.:	Date Of Issue	Page
QM17RDW005	R03	1404/03/13(03/June /2025)	1 of 2

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A. Intended use

NovaClean is a cold concentrated disinfectant solution based on peracetic acid. It is designed for the disinfection and descaling of hemodialysis machines.

B. Advantages

- Nova Clean is effective against a wide range of microorganisms (including: fungi EN1650, bacteria EN1276, and viruses ISIRI16676, ISIRI17981).
- Compatible with all peracetic acid-based hemodialysis machines; does not cause corrosion or damage.
- Biodegrades into water, Oxygen and Carbon dioxide after use; environmentally friendly.
- Easily rinsed from dialysis circuits with no toxic or allergenic residues.
- Approved by the Iranian Food and Drug Administration (IFDA) and certified for GMP compliance.

C. Compositions

Peracetic acid, Hydrogen peroxide, Acetic acid, Deionized water.

D. Warnings

- Use within 30 days after opening the container.
- Do not mix with products containing sodium hypochlorite.
- Store in a tightly closed container in a cool, well-ventilated area at 4°C to 30°C, away from direct sunlight.
- Use protective gloves and face masks when handling the solution.
- Avoid inhalation, ingestion, and direct contact with the concentrated solution.
- In case of skin or eye contact, rinse immediately with plenty of water and seek medical attention if necessary.

E. Instruction for use

To disinfect the hemodialysis machine using NovaClean, Follow the machine manufacturer's instructions for cold disinfection with peracetic acid-based solutions, or consult certified service or maintenance experts.

F. Manufacturer

Name: Nova Pharm Chemia (Member of the Iranian Iranian Consortium of Dialysis and JAM TRITA Group)

Factory: 4th Bustan, 4th Golestan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN

Office: 5th floor, No. 44, Saadat Abad Ave., Tehran, IRAN, Post code:1998836841.

Tel: +98(21)220 73102 Fac.: +98(26)37779581

Email: info@Novapharmed.co

Website: www.Novapharmed.co

G. Symbols

	Store between 4°C – 30°C
	Attention to cautions
	Read Instructions for use
	Sing up
	NFPA for NovaClean
	Health Hazard/ irritation
	Corrosive substance
	Oxidizing agent
	Manufacturer
	Keep away from sunlight
	Keep dry
	Do not open with sharp object

Doc. Code	Rev. No.:	Date Of Issue	Page
QM17RDW005	R03	1404/03/13(03/June /2025)	2 of 2