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| Doc. Code  | Rev. No.: | Date Of Issue              | Page   |
|------------|-----------|----------------------------|--------|
| QM17RDW004 | R02       | 1404/03/13(03/June. /2025) | 1 of 2 |

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### A. Intended use

NovaClean Plus is a cold concentrated disinfectant solution based on peracetic acid. It is designed for the disinfection and descaling of hemodialysis machines.

### B. Advantages

- NovaCleanPlus is effective against a wide range of microorganisms (including fungi EN1650, bacteria EN1276, and viruses ISIRI16676, ISIRI17981).
- Compatible with all peracetic acid-based hemodialysis machines, does not cause corrosion or damage.
- Biodegrades into water, Oxygen and Carbon dioxide after use; environmentally friendly.
- Easily rinsed from dialysis circuits with no toxic or allergenic residues.
- Approved by the Iranian Food and Drug Administration (IFDA) and certified for GMP compliance.

### C. Compositions

Peracetic acid, Hydrogen peroxide, Acetic acid, Deionized water.

### D. Warnings

- Use it within 30 days after opening the container.
- Do not mix with products containing sodium hypochlorite.
- Store in a tightly closed container in a cool, well-ventilated area at 1°C to 30°C, away from direct sunlight.
- Use protective gloves and face masks when handling the solution.
- Avoid inhalation, ingestion, and direct contact with the concentrated solution.
- In case of skin or eye contact, rinse immediately with plenty of water and seek medical attention if necessary.

### E. Instruction for use

To disinfect the hemodialysis machine using NovaCleanPlus, Follow the machine manufacturer's instructions for cold disinfection with peracetic acid-based solutions, or consult certified service or maintenance experts.

### F. Manufacturer

Name: Nova Pharm Chemia (Member of the Iranian Consortium of Dialysis and JAM TRITA Group)

Factory: 4<sup>th</sup> Bustan, 4<sup>th</sup> Golestan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN

Office: 5<sup>th</sup> floor, No. 44, Saadat Abad Ave., Tehran, IRAN, Post code:1998836841.

Tel: +98(21)220 73102 Fac.: +98(26)37779581

Email: [info@Novapharmed.co](mailto:info@Novapharmed.co)

Website: [www.Novapharmed.co](http://www.Novapharmed.co)

### G. Symbols

|                                                                                     |                                |
|-------------------------------------------------------------------------------------|--------------------------------|
|    | Store between 1°C – 30°C       |
|    | Attention to cautions          |
|    | Read Instructions for use      |
|    | Sing up                        |
|    | NFPA for Nova Clean Plus       |
|    | Health Hazard/ irritation      |
|    | Corrosive substance            |
|   | Oxidizing agent                |
|  | Manufacturer                   |
|  | Keep away from sunlight        |
|  | Keep dry                       |
|  | Do not open with sharp object. |

| Doc. Code  | Rev. No.: | Date Of Issue             | Page   |
|------------|-----------|---------------------------|--------|
| QM17RDW004 | R02       | 1404/03/13(03/Jun. /2025) | 2 of 2 |