1. Intended Purpose

RenaCart (polypropylene sodium bicarbonate powder cartridge), RenaCart B (polyethylene sodium bicarbonate powder cartridge), RenaBag and RenaBag S (polyethylene/polyamide sodium bicarbonate powder bag) contain 550 gr, 650 gr, or 750 gr of dry sodium bicarbonate powder of pharmaceutical grade, complying with the requirements of pharmacopoeias (USP), (Ph. Eur.) and (BP). These products are single-use and non-sterile, designed for use in hemodialysis machines for the preparation of dialysis fluid. By supplying bicarbonate buffer, these products help maintain acid-base balance during hemodialysis and related treatments.

2. Indication

It is used in the treatment of patients with acute or chronic renal failure when hemodialysis is prescribed by a physician.

3. Contraindications

There are no special contraindications for the use of Sodium bicarbonate powder for hemodialysis. Sodium Bicarbonate Powder must be used only on the directions of a physician who has evaluated all the pertinent features of the product in relation to individual patient.

4. Side effect

There is no side-effect reported for sodium bicarbonate powder. However, some undesirable effects related to the dialysis process, such as hypotension, nausea, vomiting, and muscle cramps, may occur.

5. Warning and Cautions

1. This product is single use.
2. This product is non-pyrogenic and non-sterile.
3. Do not use if the bag or cartridge is damaged or suspected to be compromised.
4. This product must only be used with the Concentrated Hemodialysis Acidic Solution recommended in the dialysis machine manual.
5. Do not use after the expiry date stated on the packaging.
6. Do not initiate dialysis without first consulting the operator's manual of the dialysis machine and performing any required tests to confirm the proper composition of the dialysis fluid.
7. Do not make unauthorized modifications to the bicarbonate cartridge, bag or to the dialysis machine, as this may lead to serious consequences, affect the therapeutic results of hemodialysis, or impair the device's function.
8. This product must only be used by trained nurses or physicians who are fully familiar with the procedure and strictly follow the instructions for use.
9. Pharmed Medical Industries is not responsible for issues resulting from improper storage or transportation, misuse, or use with incompatible devices.
10. Dispose the product after use, according environment health regulations.
11. The solution prepared from Sodium Bicarbonate Powder should be consumed immediately after preparation.
12. Use only quality water that meets the requirements of BS EN ISO 23500-3 to dilute the concentrate for preparing dialysis fluid.
13. Sodium Bicarbonate bag and cartridge which are suitable for all acidic formulas as per the ration stated on the label of container of concentrated hemodialysis solution.
14. Acid and basic bicarbonate hemodialysis concentrates should only be diluted for short time before use.
15. The composition of the final dialysis fluid for use must be closely monitored.
16. For single dialysis session use only. Bicarbonate solution is highly susceptible to microbial contamination; therefore, dispose any unused solution.

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6.Direction for use

Sodium bicarbonate bags and cartridges contain 550 g, 650 g, or 750 g of sodium bicarbonate powder, designed to dissolve in 4 to 5 liters of purified water. This solution is then mixed in the dialysis machine with acid concentrate and water to produce the final dialysate with a flow rate of 500 ml/min and a sodium bicarbonate concentration of approximately 35 mEq/L. Each cartridge or bag is sufficient for one 4 to 5 hour dialysis session (according to the amount of powder).

For using these products, please refer to the dialysis machine manufacturer's instructions for proper installation, operation, and disassembly procedures.

7.Type of Hemodialysis machine

RenaCart

RenaCart can be used with the following dialysis machines:
Nipro (Sordial 55 plus)
Bellco (formula 2000, formula Therapy, Formula Plus)
Hospal (integra, monitral),
Gambro (AK95, AK200, AK96) B-Brun (Dialog, Dialog+)
Renova

RenaCart B

RenaCart B can be used with the following dialysis machines:
JMS (SDS 20, SDS 50), IDEMSA

RenaBag and RenaBag S

RenaBag shall be use with Fresenius (4008 B) and RenaBag S shall be use with Fresenius (4008 S, 5008) dialysis machines.

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Note: RenaCart B Products haven't the CE mark.

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Nipro (Sordial 55 plus)
Bellco (formula 2000, formula Therapy, Formula Plus)
Hospal (integra, monitral)
Gambro (AK95, AK200, AK96), B-Brun (Dialog, Dialog+)
Renova

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9.Storage Condition

Store at 5°C to 30°C in a dry place, protected from direct sunlight and severe vibration. Protect from freezing.

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NephroCan GmbH. & Co.KG